



Griff® DISPO Sharps container


Assembly and usage instructions

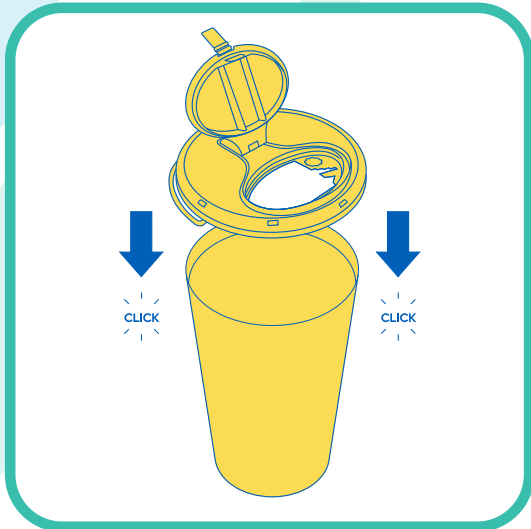


- 1** Fix the lid to the base by applying firm pressure. You'll hear and feel clicks as the lid fixes into place.

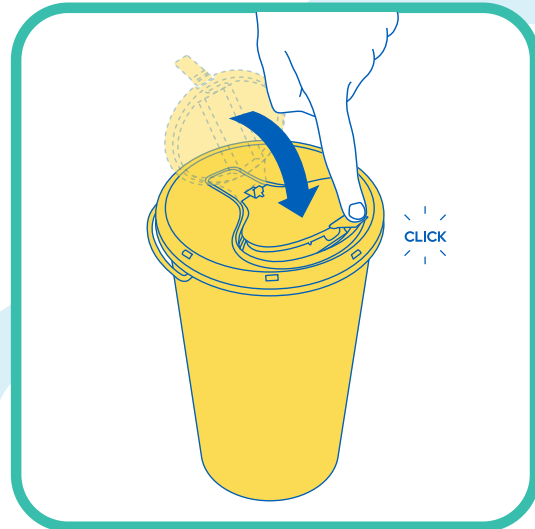

- 2** For temporary closure, press the base of the tab where it joins the lid as shown and apply light downwards pressure until you hear and feel a click.

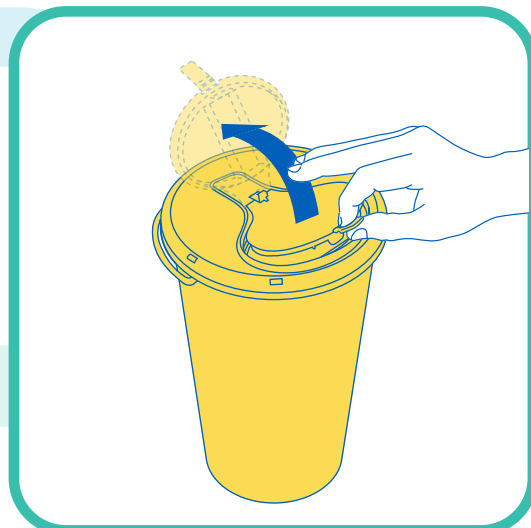

- 3** To release temporary closure, pull the tab upwards.

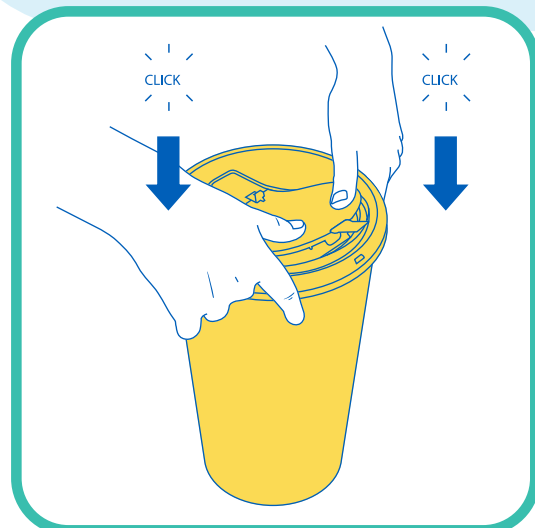

- 4** To permanently seal, push firmly at the two points indicated until clicks are heard and felt. A final visual check is recommended.

**Do not fill above the indicated line.
Complete the relevant sections of the label on the container.**